

We're specialists in fixation products.

Find out more about **Optimum Medical** products, including our **Ugo Fix** range, by visiting:

www.optimummedical.co.uk



optimummedical

Optimum Medical

First floor, Unit 5, Armley Court, Armley Road,
Leeds, LS12 2LB, United Kingdom

Tel.: +44 (0) 8456 435 479, Fax: +44 (0) 1142 383 826

www.optimummedical.co.uk



Company Registration No: 6779883 OMS-UFBB-100 Oct 16

multifixTM
gentle

universal
securement device
for medical tubings


with **silicone technology**
for atraumatic patient experience



optimummedical®



universal securement device
for medical tubings

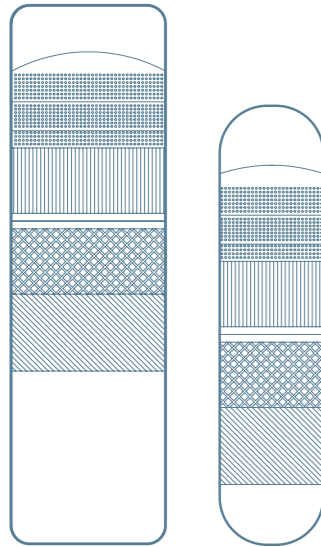

large



small



up to 7 days

Secure, reliable and gentle.

The **Multi Fix Gentle** is a universal securement device for medical tubings and urinary catheters. It has been developed to provide comfortable fixation, giving the patient the confidence to move around freely.

Multi Fix Gentle is available in 2 sizes to suit a range of applications, and each one can be used for up to 7 days.

'Proper securement is a priority'¹

The **Multi Fix Gentle** benefits from **soft silicone technology** to ensure it's easy to apply, reposition and there is minimal trauma when it's removed.

SOFT SILICONE RE-ADHESIVE GEL STRIP

Because '...Some patients suffer from impaired skin integrity that makes stabilization more challenging'²


Multi Fix Gentle has been designed to conform to the skin with a soft silicone re-adhesive gel strip which provides a range of patient-focused benefits including:

- Atraumatic removal from the skin without use of solutions to dissolve adhesive
- No sticky residue left behind
- Conforms to all skin types through multiple points of contact
- Can be repeatedly applied to same area of skin
- Re-adhesive tack enables easy repositioning
- No complex skin preparation required (no need to shave hairy skin before application)
- Reduced risk of allergic reactions due to silicone technology
- Showerproof, breathable and translucent

OFFERING EFFECTIVE TUBE FIXATION

Suitable for adhesive fixation of a range of medical tubings

- Able to remove and reposition tubing
- Leaves no residue on tubings
- Can accommodate range of tubing diameters

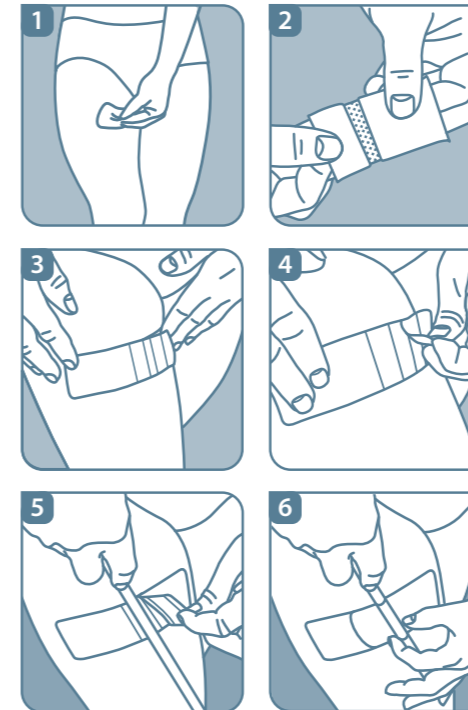

INSTRUCTIONS FOR USE

- 1 • Ensure the skin is clean and dry (there is no need to prepare the skin or shave)
- 2 • Remove the backing from the **Multi Fix Gentle**
- 3 • Place the **Multi Fix Gentle** onto the skin in the desired spot and smooth in place
- 4 • Lift the front tab on the **Multi Fix Gentle** strip and remove the release paper to reveal the adhesive
- 5 • Place the catheter or tube on the base pad and replace the adhesive tab back in place over the catheter or tubing to secure
- 6 • Ensure your catheter or tubing is not pulling and you are able to move around comfortably with it in place.

Should you need to reposition the **MultiFix Gentle** simply peel off and replace on the skin. Additionally, the tubing can also be repositioned as necessary.

OM Code	Product Description	Size	Inner Pack	Outer Pack
3030	Multi Fix Gentle (large)	140X40mm	Box of 10	Case of 200
3031	Multi Fix Gentle (small)	110X25mm	Box of 10	Case of 200

REFERENCES

- ¹ Bell M A (2010) Severe Indwelling Urinary Catheter-Associated Urethral Erosion in four elderly men. *Ostomy and Wound Management*. December 2010 pp36-39
- ² Callan L. & Parker D. et al. (2012) Wound, Ostomy and Continence Nurse Society, Clinical Practice Continence Committee. *Indwelling Urinary Catheter Securement: Best Practice for Clinicians*; Available at <http://www.faet.org/docs/indwellingurinarycathetersecurement.pdf>