SWE

Multi Fix Gentle

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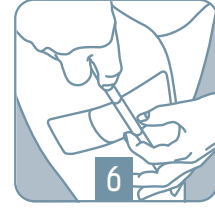
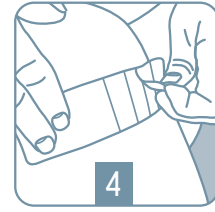
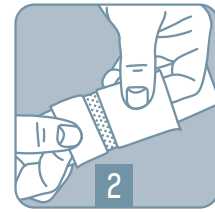
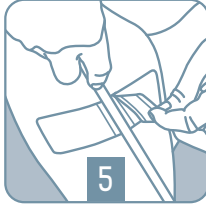
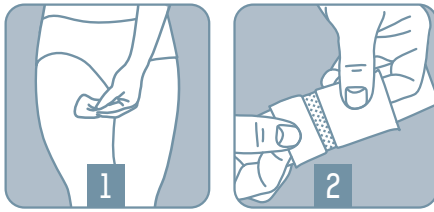
GBR

Multi Fix Gentle

Atraumatic soft silicone re-adhesive gel strip for universal tube fixation

At Optimum Medical we understand the need for secure and reliable fixation of medical tubing and urinary catheters. We have developed the Multi Fix Gentle universal securement device to provide comfortable fixation, giving the patient the convenience to move around freely.

Benefits of using the Multi Fix Gentle

The Multi Fix Gentle is available in 2 sizes and can be used for up to 7 days. Only change it sooner if it is damaged or soiled. You can shower while wearing the Multi Fix Gentle, however exposure to water and higher levels of activity may mean that you need to change the device more frequently.

Soft silicone re-adhesive gel strip

- Atraumatic removal without use of solutions to dissolve adhesive
- No sticky residue
- Conforms to all skin types through multiple points of contact
- Can be repeatedly applied to same skin area
- Re-adhesive tack enables easy repositioning
- No complex skin preparation required

Instructions for use

1. Ensure the skin is clean and dry (there is no need to prepare the skin or shave)
2. Remove the backing from the Multi Fix Gentle
3. Place the Multi Fix Gentle onto the skin in the desired spot and smooth in place
4. Lift the front tab on the Multi Fix Gentle strip and remove the release paper to reveal the adhesive
5. Place the catheter or tube on the base pad and replace the adhesive tab back in place over the catheter or tubing to secure
6. Ensure your catheter or tubing is not pulling and you are able to move around comfortably with it in place.

Should you need to reposition the Multi Fix Gentle simply peel off and replace on the skin. Additionally, the tubing can also be repositioned as necessary.

CODE	PRODUCT	SIZE	INNER BOX	OUTER BOX
3030	Multi Fix Gentle	Large 140x40mm	10/box	200/case
3031	Multi Fix Gentle	Small 110x25mm	10/box	200/case

WARNING: Keep out of reach of children!

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