

ugo.<sup>TM</sup>  
leg bag

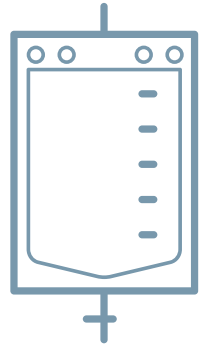STERILE EO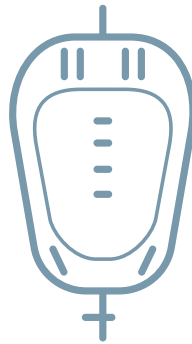

2797  
STERILE

NON STERILE

Ugo-leg-bags-FEU-Feb20-V4.0

optimummedical®

**Optimum Medical Solutions Ltd.**  
Tennant Hall, Blenheim Grove,  
Leeds, LS2 9ET, United Kingdom,  
Tel.: +44 (0) 8456 435 479  
enquiries@optimummedical.co.uk  
www.optimummedical.co.uk

ugo.<sup>TM</sup>  
2L drainage bag

# Instructions for use

# UGO LEG BAGS AND UGO 2L DRAINAGE BAGS

## INSTRUCTIONS FOR USE

**Ugo Leg Bag** and **Ugo 2L Drainage Bag** are for the collection, storage and drainage of urine.

### UGO LEG BAGS

The **Ugo Leg Bag** can remain attached to your catheter for up to 7 days. All leg bags are sterile pouch packaged and have a needle free sample port. Supplied in boxes of 10.

**Never reconnect a used leg bag.**

### COMMUNITY LEG BAGS - fabric backing

- 1 pair of **Ugo Fix Leg Bag Straps** in every box

REF.	Product description			
1C/500/ST/L	<b>Ugo 1C</b>	500ml	Short tube (5cm)	lever tap
2C/500/LT/L	<b>Ugo 2C</b>	500ml	Long tube (25cm)	lever tap
3C/500/ST/T	<b>Ugo 3C</b>	500ml	Short tube (5cm)	T tap
4C/500/LT/T	<b>Ugo 4C</b>	500ml	Long tube (25cm)	T tap
5C/350/DI/L	<b>Ugo 5C</b>	350ml	Direct inlet	lever tap
6C/350/ST/L	<b>Ugo 6C</b>	350ml	Short tube (5cm)	lever tap
7C/750/ST/L	<b>Ugo 7C</b>	750ml	Short tube (5cm)	lever tap
8C/750/LT/L	<b>Ugo 8C</b>	750ml	Long tube (25cm)	lever tap
3410	<b>UgoPro 1C</b>	500ml	Short tube (5cm)	lever tap + pair of nitrile gloves + apron
3411	<b>UgoPro 2C</b>	500ml	Long tube (25cm)	lever tap + pair of nitrile gloves + apron
3412	<b>UgoPro 3C</b>	500ml	Short tube (5cm)	T tap + pair of nitrile gloves + apron
3413	<b>UgoPro 4C</b>	500ml	Long tube (25cm)	T tap + pair of nitrile gloves + apron

### HOSPITAL LEG BAGS - no fabric backing

- 1 pair of **Ugo Fix Leg Bag Straps** in every pouch

REF.	Product description			
1H/500/ST/L	<b>Ugo 1H</b>	500ml	Short tube (5cm)	lever tap
2H/500/LT/L	<b>Ugo 2H</b>	500ml	Long tube (25cm)	lever tap
3H/500/ST/T	<b>Ugo 3H</b>	500ml	Short tube (5cm)	T tap
4H/500/LT/T	<b>Ugo 4H</b>	500ml	Long tube (25cm)	T tap

### HOW TO ATTACH YOUR UGO LEG BAG TO YOUR CATHETER

- Wash and dry your hands (if a carer is changing the leg bag for you they must apply a clean pair of gloves after drying their hands).
- Remove any straps or sleeve which are supporting your **Ugo Leg Bag** to your leg.
- Open the pouch packaging for a new **Ugo Leg Bag** and remove the protective cap from the leg bag stepped connector.

- Detach the old leg bag from the catheter and immediately insert the stepped connector from the new leg bag ensuring an adequate seal (the leg bag is supplied in the closed position).
- It is important that you do not touch the ends of the catheter or leg bag connector during the change (non touch technique).
- The discarded protective cap from the new **Ugo Leg Bag** can be attached to the stepped connector of the old leg bag.
- Apply your leg bag fixation and support devices and adjust to ensure optimum drainage and comfort. If using the **Ugo Fix Leg Bag Straps** provided, feed the leg straps through the eyelets on the bag ensuring the silicone grip faces onto the leg. Alternatively you can use the **Ugo Fix Sleeve** for extra support. To find out more contact [ugo@optimusmedical.co.uk](mailto:ugo@optimusmedical.co.uk).
- Empty the urine from your old leg bag into the toilet and dispose of the bag as recommended by your nurse.
- Wash and dry your hands.

### HOW TO EMPTY YOUR UGO LEG BAG

- Wash and dry your hands (if a carer is emptying your bag for you they must apply a clean pair of gloves after drying their hands).
- Hold the outlet tap over a toilet or suitable receptacle.
- Open the tap by sliding the T tap or pushing the lever tap down.
- Allow the bag to drain completely then close the tap.
- Dry the end of the soft silicone outlet tubing with a clean wipe if necessary.
- Wash and dry your hands.

### UGO 2L DRAINAGE BAGS

All the **Ugo 2L Drainage Bags** are suitable for link drainage (attaching to your leg bag for additional overnight drainage).

**Ugo 9, Ugo 10, Ugo 10 Xtra, and Ugo 11** can only be drained once. **Ugo 9, Ugo 10** and **Ugo 10 Xtra** are suitable for link drainage only and cannot be attached directly to a catheter. **Ugo 11** is suitable for link drainage and attaching directly to a catheter.

**Ugo 12** and **Ugo 13** have a needle free sample port and are suitable for attaching directly to a catheter for up to 7 days.

### UGO 2L Drainage Bags - 90cm Inlet Tubing

REF.	Product description		Quantities
9/NS/SU/DO	<b>Ugo 9</b>	non drainable/tear strip non sterile	resealable bag of 10
10/NS/SU/DO	<b>Ugo 10</b>	single use T tap non sterile	resealable bag of 10
11/S/SU/DO	<b>Ugo 11</b>	single use T tap sterile pouch packaging	box of 10
12/S/D/L	<b>Ugo 12</b>	lever tap sterile pouch packaging	box of 10
13/S/D/T	<b>Ugo 13</b>	T tap sterile pouch packaging	box of 10

### Ugo Xtra 2L Drainage Bags - 120cm Inlet Tubing

REF.	Product description		Quantities
3300	<b>Ugo 10 Xtra</b>	single use T tap non sterile	resealable bag of 28

## HOW TO ATTACH AND REMOVE YOUR UGO 2L DRAINAGE BAG AS PART OF A LINK DRAINAGE SYSTEM

If you need additional drainage overnight then the **Ugo 2L Drainage Bag** can be connected to the soft silicone outlet tube on the end of your **Ugo Leg Bag** (this is called link drainage).

### BEDTIME - ATTACHING THE UGO 2L DRAINAGE BAG

- Wash and dry your hands (if a carer is doing this for you they must apply a clean pair of gloves after drying their hands).
- Remove any straps or sleeve which are supporting your **Ugo Leg Bag** to your leg.
- Remove the protective cap from the **Ugo 2L Drainage Bag** stepped connector.
- Insert the stepped connector from the **Ugo 2L Drainage Bag** into the soft silicone tubing on the **Ugo Leg Bag** (ensure an adequate seal).
- Attach the **Ugo 2L Drainage Bag** to a stand.
- Open the lever tap or T tap valve on your **Ugo Leg Bag** to allow immediate and continuous drainage.
- Wash and dry your hands.

### MORNING - REMOVING THE UGO 2L DRAINAGE BAG

- Wash and dry your hands (if a carer is doing this for you, they must apply a clean pair of gloves after drying their hands).
- Close the lever tap or T tap on your **Ugo Leg Bag**.
- Disconnect the **Ugo 2L Drainage Bag** from the soft silicone tubing on the **Ugo Leg Bag**.
- Apply your leg bag fixation and support devices and adjust to ensure optimum drainage and comfort.
- Empty the urine from the **Ugo 2L Drainage Bag** into the toilet and dispose of bag as recommended by your nurse.
- Wash and dry your hands.
- Never reconnect a used 2 litre drainage bag.

## PRECAUTIONS

For single use only, discard after use. If product is re-used, may result in infection.

Do not use if packaging is damaged.

If your urinary drainage system is not draining urine and/or you are in discomfort or feeling unwell, contact your nurse or doctor immediately.

Do not store in toxic, harmful or corrosive conditions.

5 year shelf life.

Please note volume markings printed on the bags are approximate.

Do not use for accurate recording of urine output.

**WARNINGS:** Keep out of reach of children.  
Risk of suffocation. Please keep plastic bags away from small children and babies.

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